




Atrauman Ag[®]:
the versatile, non-adherent,
triglyceride-impregnated silver
dressing with antibacterial
properties.

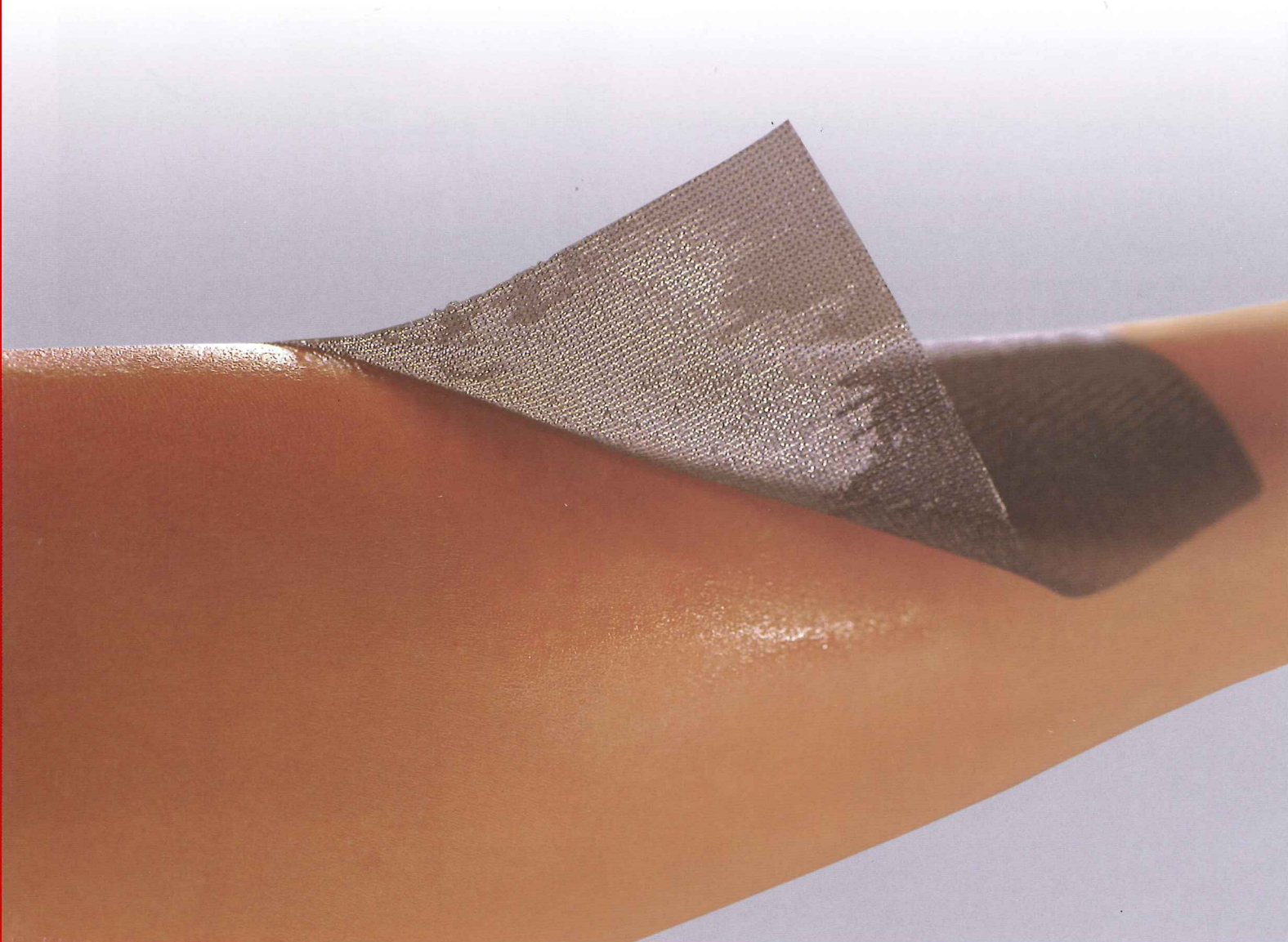



A versatile, economic, impregnated, silver dressing.

What is Atrauman Ag used for?

Atrauman Ag is used for atraumatic treatment of colonised, critically colonised or infected wounds. It is effective against both Gram-negative and Gram-positive bacteria, including methicillin resistant *Staphylococcus aureus*.

- Abrasions and lacerations.
- Leg ulcers.
- Pressure ulcers.
- Burns and scalds.

Atrauman Ag can also be used for infection prophylaxis.

Atrauman Ag: the benefits.

- Ionic silver produces antimicrobial action.
- May be used to prevent infection.
- Effective for at least 7 days.
- Soft and conformable to ensure intimate contact with wound.
- Triglyceride impregnation and fine weave mesh minimise adhesion.
- No skin discolouration.
- Allows easy passage of exudate away from wound surface.
- Non-petroleum-based triglyceride leaves no wound residues.
- Impregnation protects wound edges, helping to prevent maceration.

Case study 1

Presentation

80 year old patient presented following toe resection with a wound 3.5 x 2 cm diameter. The wound was infected and was covered with 40% granulation tissue and 40% epithelial tissue. There was some exudation.

Treatment

The wound bed was treated with Atrauman Ag for seven weeks. PermaFoam® was used as a secondary dressing and initially changed daily. The patient was given amoxicillin for the infection.

Assessment 1


After 4 days of treatment with Atrauman Ag, the infection had abated so the course of amoxicillin was stopped.

Assessment 2

After 2 weeks the wound was completely clean and free of coatings and 50% of the wound area was either epithelised or granulated.

Conclusion

Patient tolerated treatment with Atrauman Ag very well. After 7 weeks of treatment the wound treatment was successfully concluded.

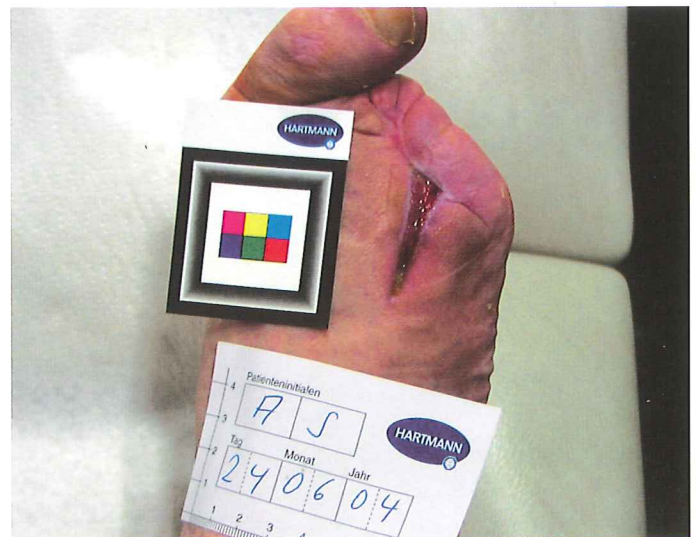

	Unit		Code No	Units per case
Sterile individually sealed	5 x 5 cm	10 pcs	499 571	6
	10 x 10 cm	10 pcs	499 573	6
	10 x 20 cm	10 pcs	499 575	6



THE HARTMANN ALLIANCE

NEW SOUTH WALES

Paul Hartmann Pty Ltd
Phone: 1800 805 839

VICTORIA & TASMANIA

Sutherland Medical Pty Ltd
Phone: 1300 664 027

QUEENSLAND

Medical & Surgical Requisites Pty Ltd
Phone: 1800 801 226

SOUTH AUSTRALIA & NT

McNeil's Surgical Pty Ltd
Phone: 08 8363 0888

WESTERN AUSTRALIA

Perth Surgical Supply Co. Pty Ltd
Phone: 08 9344 3111

NEW ZEALAND

USL Medical Ltd
Phone: 0800 658 814