

# Hydrosorb Gel

The preservative free hydrogel that donates moisture, softens necrotic and devitalised tissue thereby facilitating wound healing





# Hydrosorb Gel

The preservative free hydrogel that donates moisture, softens necrotic and devitalised tissue thereby facilitating wound healing

## Indications for use

- Rehydration and debridement of slough and necrotic tissue
- To provide a moist environment in wounds of various aetiology
- Pain relief

## Product properties and action

Hydrosorb® Gel is a clear, viscous, sterile gel which is used for the treatment of dry chronic wounds. It contains Ringers solution, glycerol, hydroxy-ethyl cellulose, carboxy-methyl cellulose. It does not contain polyethylene glycol.

Hydrosorb® Gel provides a moist wound environment helping to promote wound healing. It softens dry necrotic tissue, while facilitating the removal of devitalised tissue and absorbing wound debris and exudate.

The syringe presentation enables gel application to be delivered directly into the wound for safe, clean dosing. The reverse scale plunger allows accurate measuring of dispelled contents to the nearest millimetre, thus minimising the risk of application errors and enabling simple documentation.

## Application

The gel may be left on a wound for up to three days and requires a suitable secondary dressing to prevent drying out

## Contraindications/precautions

- Full thickness burns



| CODE  | SIZE | UNIT  |
|-------|------|-------|
| 11205 | 8g   | 5/box |
| 1048  | 15g  | each  |