

ProShield® N-95

Particulate Filter Respirator,
Fluid Resistant

APPLICATION GUIDE



- Wash hands or use an alcohol based hand gel as an option for disinfection.
- Separate mask to open (stamp side down).
- Gently curve the nosepiece to conform respirator to your nose.



- With your index fingers and thumbs, separate the two headbands.



- While holding the headbands with your index fingers and thumbs, place your chin in the lower part of the respirator.
- Pull the headbands over your head.
- Release the upper headband from your fingers and position it at the crown of your head.



- Release the remaining headband and position it at the base of the neck.



- Whilst holding the nosepiece at the ends with your thumbs, lightly mould along the entire nosepiece with your fingers to ensure a facial fit.
- Continue to adjust the respirator until you feel you have achieved a good facial fit and perform a fit check (a user seal check).

PROSHIELD® N-95 PARTICULATE FILTER RESPIRATOR

A respirator is a respiratory protective device designed to protect the wearer against inhalation of contaminated air.

PROSHIELD® N-95 RESPIRATORS ARE:

- NIOSH certified
- P2 compliant
- Fluid resistant to 160mmHg
- Suitable for sensitive skin
- Available in 2 sizes



Fit Checking is a procedure to assist in determining whether the wearer has donned the respirator correctly and should be performed each time a respirator is donned or adjusted to ensure proper sealing of the respirator to the face.

AS/NZS 1716: 2012. Respiratory Protective Devices

DIRECTIONS FOR CHECKING THE FACIAL SEAL OF PROSHIELD® N-95 PARTICULATE FILTER RESPIRATOR

- To Fit Check **ProShield® N-95**, forcibly inhale and exhale, whilst the respirator is in place.
- The respirator should collapse slightly on inhalation and expand on exhalation.
- To obtain a good facial fit and a successful facial check, you should not be able to feel any air leakage between the respirator and your face.
- If this does not occur, please adjust and respirator until leakage is corrected and a successful facial seal is obtained.
- If experiencing difficulties in obtaining a successful Fit Check, utilisation of a mirror can be of assistance. Ask someone to look for hair or earrings that might be caught in the seal.
- Ensure headbands are positioned correctly with the top headband positioned at the crown of your head, as it is designed to hold the bottom of the respirator securely against your chin.
- The wearer should not proceed with any activities until a successful Fit Check has been completed.

Note: The use of respirators with tight fitting facepieces (N-95) is not permitted to be worn by individuals with facial hair as it will interfere with the seal of the respirator.

THE REMOVAL (DOFFING) OF PROSHIELD® N-95

- Wash hands or use an alcohol based hand gel as an option for disinfection.
- The front of the respirator is considered contaminated, handle only the bands.
- To remove, pull lower band over the head first, then remove upper band.
- Discard in an appropriate receptacle making sure the headbands are the only part in touch with your hands.
- Wash hands.



Ref-no.	Item Number	Description	Items per unit	Shipper Qty
72509-09	502354	ProShield® N-95 Small	bx/50	6 bxs
72509-10	502355	ProShield® N-95 Medium	bx/50	6 bxs

THERAPIES. HAND IN HAND.

www.bsnmedical.com.au www.bsnmedical.co.nz
©Registered Trademark

BSNmedical

an Essity company