

## Table of content

<b>SIGVARIS MEDICAL assortment</b>	2	<b>CUSTOM MADE assortment</b>	41
<b>CORE assortment</b>		<b>Alterations</b>	46
▪ MAGIC	8	<b>Donning aids</b>	
▪ COMFORT	12	▪ DOFF N' DONNER (DND)	49
▪ COTTON	14	▪ SIGVARIS sim-slide	50
<b>EXTRA assortment</b>		▪ SIGVARIS magnide on/off	50
▪ JAMES – High tech and style	18	▪ Rubber gloves	51
▪ TRADITIONAL – For the highest medical standards	20	▪ Foot slips	51
▪ Calf with grip top	22	<b>Care products</b>	52
▪ Larger foot sizes	23	<b>Pressure values / Contraindications</b>	54
▪ Half-thigh	24		
▪ Thigh with soft top	25		
▪ Thigh with waist attachment	26		
▪ Short lengths	27		
▪ Pantyhose with compression in panty part	28		
▪ Venous ulcers treatment – ULCER X	29	<b>Sizing systems</b>	
▪ Post-surgery	30	▪ Compression arm sleeves	56
▪ Compression classes 3 and 4	32	▪ TRADITIONAL, ULCER X	57
▪ Thrombosis prophylaxis	36	▪ MAGIC, COMFORT, COTTON, JAMES	60
▪ Compression arm sleeves	38		

# SIGVARIS MEDICAL

SIGVARIS is the specialist for circular knit compression stockings. We offer an extensive selection of medical compression stockings. To help you find what you are looking for quickly and easily, we have divided our products into **Core, Extra and Custom Made**:

## Core assortment

Our Core assortment provides you with a complete selection of products from the MAGIC, COMFORT and COTTON lines. You can select from the following model combinations:

- Product lines: MAGIC, COMFORT or COTTON
- Models: calf-length stocking, thigh-length stocking, pantyhose and pantyhose maternity
- Compression classes: 1 or 2
- Toe: open or closed
- Sizes: 12 sizes from XXS to XL PLUS

## Extra assortment

The Extra assortment offers solutions for your patient's individual needs. To help you find the right product as quickly as possible, they are listed by customer requirements:


- JAMES – High tech with style for men
- TRADITIONAL – For the highest medical standards
- TOP FINE SELECT – Smart product range
- Special rim types, short models, bodyform designs and solutions for large feet
- Compression stockings classes 3 and 4
- Thrombosis prophylaxis and arm sleeves



### **Custom Made assortment**

In our Custom Made assortment each product is based on the patient's individual measurements.

Each item is made to order – with the customary SIGVARIS «Made in Switzerland» standard and short delivery times guaranteed.

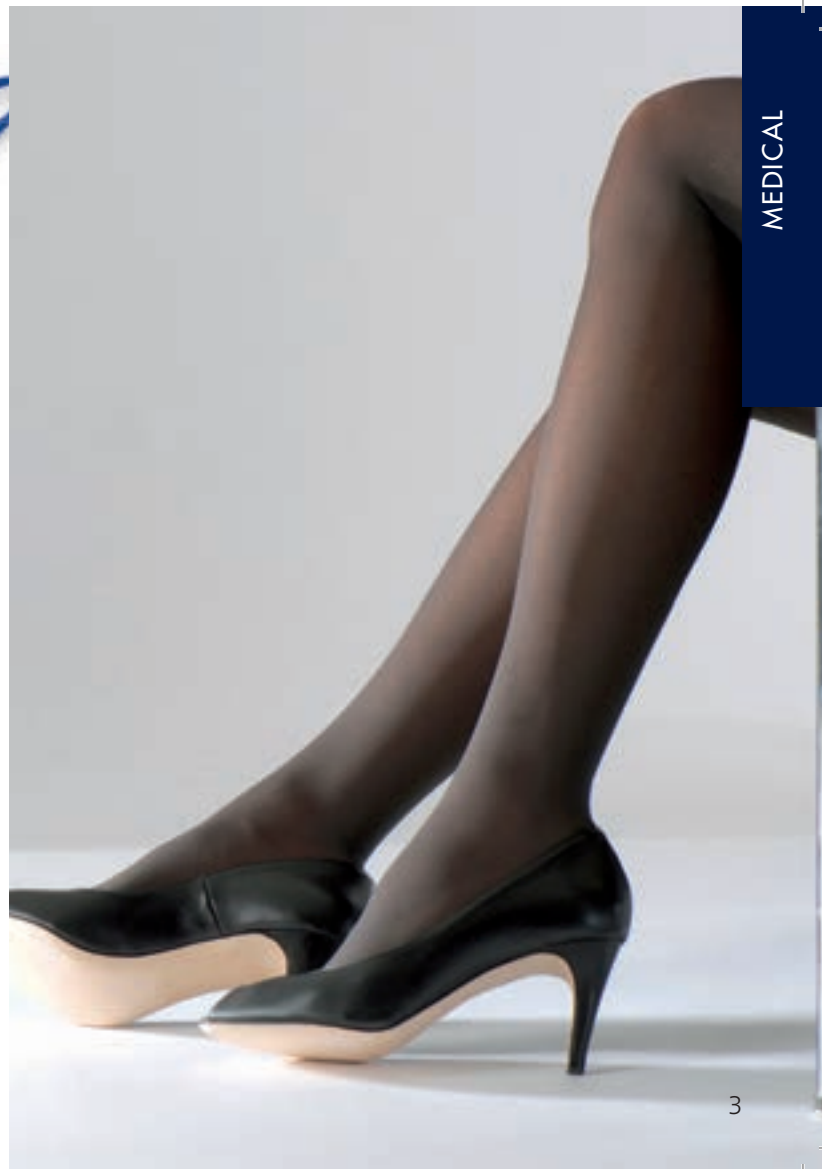

## Grip tops

### MAGIC

Sensinova grip top SVGT



Quadrinova knobbed grip top KGT



### COMFORT / COTTON

Sensinova grip top SVGT



Quadrinova knobbed grip top KGT



### TRADITIONAL

Knobbed grip top KGT







SIGVARIS

SIGVARIS

# SIGVARIS MEDICAL

The CORE assortment



SIGVARIS



## MAGIC





- For women
- Fine and elegant
- Semi-sheer finish
- Provides optimum support for the legs and bottom part
- 11 colours





## MAGIC – For the highest aesthetic standards

The elegant look and subtle sheer quality of MAGIC appeals to the senses. Particular attention has been paid to the aesthetics of this range. This becomes evident in the feminine elastic rims, the «push-up» effect for the bottom part and the unique Sensinova grip top design.

CCL 1/2 Open and closed toe	A-D	A-G		A-T	A-T Maternity	Colours
						
	Soft top	Sensinova grip top	Quadrinova knobbed grip top	Soft top	Soft top	
<b>MAGIC</b>	✓	✓	✓	✓	✓	<div> <div></div> savanna 97           <div></div> skin 81           <div></div> caramel 25           <div></div> black 99         </div> <hr/> <div>           No stock item, produced on order:           <div></div> brown 11           <div></div> grey 14           <div></div> marine 10           <div></div> coral 43           <div></div> wave 41           <div></div> pearl 40           <div></div> deep sea 42         </div>

MAGIC in the EXTRA assortment:

- Calf with grip top (p. 22)


SIGVARIS




## MAGIC COLOURS 2015/2016

Coral, Wave, Pearl, Deep Sea – the four trend colours from SIGVARIS capture the main trends of the 2015/2016 fashion season. The seasons are distinguished by statement colours, expressive art prints, metallic and sea shades in all tones and depths, as well as flattering nude tones and lots of black and white. Colour shading and graduation make for very versatile pieces, from tonal brightness to infinite depth of colour.

The current MAGIC COLOURS are right on point with the latest trends.

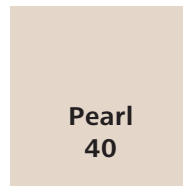

SIGVARIS

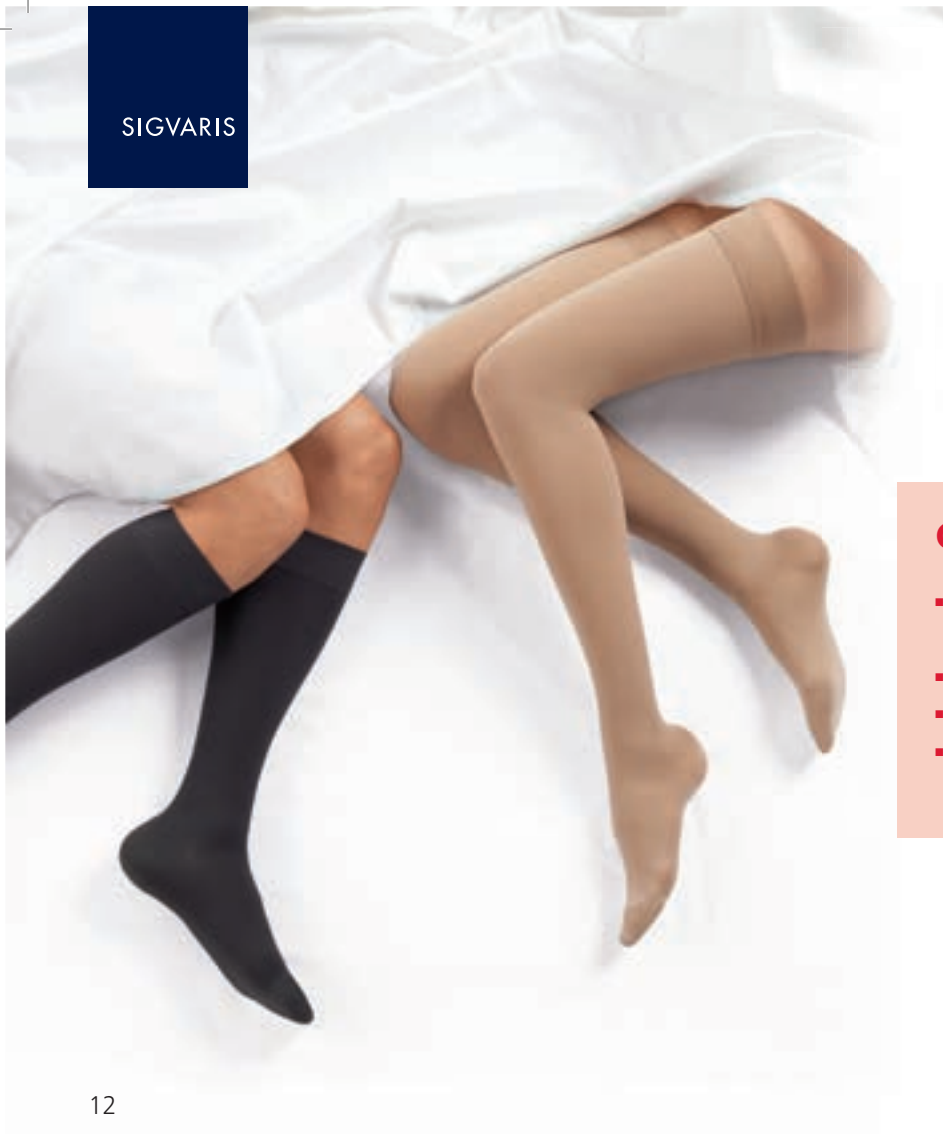





## COMFORT

- Provides the perfect snug fit for all shapes
- Uncomplicated to handle
- Opaque finish
- Conceals venous problem zones



## COMFORT – For the highest wearing comfort

COMFORT medical compression stockings are suitable for every occasion. Thanks to their soft knit and optimal fit they are very comfortable to wear, making COMFORT ideal for women and men alike. COMFORT stockings are uncomplicated to handle and adapt perfectly to any shape.

CCL 1/2 Open and closed toe	A-D	A-G		A-T	A-T Maternity	Colours
						
	Soft top	Sensinova grip top	Quadrinova knobbed grip top	Soft top	Soft top	
COMFORT	✓	✓	✓	✓	✓	<div></div> savanna 97 <div></div> skin 81 <div></div> caramel 25 <div></div> grey 14 <div></div> black 99

### COMFORT in the EXTRA assortment:

- Calf with grip top (p. 22)
- Thigh with waist attachment (p. 26)
- Thigh with soft top (p. 25)
- Bodyform (p. 28)





SIGVARIS





## COTTON

- For women and men with sensitive skin
- Excellent thermoregulation
- Opaque with a subtle ribbed texture
- Optimal mix of fibres means that the stocking slides on and off easily



## COTTON – For the highest functional requirements of the skin

The exclusive cotton yarn used in COTTON stockings is knitted on the inside of the stocking and is in direct contact with the skin. Thanks to cotton's natural properties, COTTON stockings are particularly suitable for customers with sensitive skin, ensuring greater comfort and well-being. The mix of fibres also ensures a balanced temperature in fluctuating climatic conditions.

CCL 1/2 Open and closed toe	A-D	A-G		A-T	A-T Maternity	Colours
						
	Soft top	Sensinova grip top	Quadrinova knobbed grip top	Soft top	Soft top	
<b>COTTON</b>	✓	✓	✓	✓	✓	<div>■ nature 76</div> <div>■ black 99</div>

### COTTON in the EXTRA assortment:

- Calf with grip top (p. 22)
- Thigh with waist attachment (p. 26)
- Larger foot sizes (p. 23)
- Short lengths (p. 27)
- Thigh with soft top (p. 25)
- Compression class 3 (p. 33)

SIGVARIS



# SIGVARIS MEDICAL

The EXTRA assortment



SIGVARIS

## JAMES




- For men
- Business and casual
- Large feet
- Subtle ribbed texture
- Reinforced heel
- Seamless toe section
- Full sole cushioning





## JAMES – High tech with style

The compression stocking JAMES represents the perfect combination of high tech and style, and works just as well with a suit or casual outfit. Thanks to the full sole cushioning and integrated odour-inhibiting zinc impregnation, your feet will stay fresh even when things heat up.

Product line	Model	 Toe 	Rim	CCL	Colours
JAMES	A-D 		closed	Soft top	2
					<div></div> cyanide 89 <div></div> carbon 88 <div></div> black 99



SIGVARIS















## TRADITIONAL – For the highest medical standards

The yarn's unique stress/strain behaviour guarantees high pressure stability throughout the day and the effective treatment of severe venous diseases. TRADITIONAL can also easily be worn in high pressure classes.

- High pressure stability
- Robust and durable
- Breathable
- Very comfortable



Available sizes (p. 57)

Product line	Model	 Toe 	Rim	CCL	Colours	
TRADITIONAL	A - B <sub>1</sub> 	open		Ankle-high	–	 beige 77
TRADITIONAL	A - D 	open		Soft top	2/3/4	 beige 77
				Knobbed grip top	2/3	
	A - F 	open		Soft top	2/3/4*	 beige 77
				Knobbed grip top	2/3	
	A - G 	open		Soft top	2/3/4*	 beige 77
				Knobbed grip top**	2/3	
				Waist attachment	2/3*	
	A - T Panythos A - T Maternity A - T for man 	open		Soft top	2*	 beige 77
	A - G Post-surgery 	open		Waist attachment incl. under stockings	2*/3*	 beige 77
















\* Only PLUS sizes

\*\* Compression up to the cG level (without soft-part)

## Calf with grip top

This alternative calf-length model is suitable in cases where the calf stocking with soft top slips. It is used on patients with straight calves (if the circumference measurements in the cC area correspond with the cD area) or if the patient has excessive leg hair.

- For greater leg support
- Can be used if calf stocking with tricot rim slips

Calf with grip top			 Toe 	Rim	CCL	Colours	
MAGIC	A-D		open	closed	Quadrinova knobbed grip top narrow	2	 savanna 97  caramel 25  black 99
COMFORT	A-D		open	closed	Quadrinova knobbed grip top narrow	2	 savanna 97  caramel 25  black 99
COTTON	A-D		open	closed	Quadrinova knobbed grip top narrow	2	 nature 76  black 99
TRADITIONAL	A-D		open		Knobbed grip top narrow	2/3	 beige 77

## Larger foot sizes

### Large feet: JAMES





This men's stocking is ideal for large feet of all sizes. JAMES has been especially designed for men's feet and its large foot can also alleviate symptoms relating to hallux valgus (bunions).

- Large ball of the foot (cA)
- Raised instep
- Long heel-to-instep diagonal (Y measurement)

### Long feet: COTTON Maxifoot

- For patients with long feet
- Unisex product



Larger foot sizes	 Toe 	Rim	CCL	Colours
<b>Large feet:</b> <b>JAMES</b> <div><div>A - D</div></div>	<div></div> <div>closed</div>	Soft top	2	<div><div></div> cyanide 89</div> <div><div></div> carbon 88</div> <div><div></div> black 99</div>
<b>Long feet:</b> <b>COTTON Maxifoot*</b> <div><div>A - D</div></div>	<div></div> <div>closed</div>	Soft top	2	<div><div></div> nature 76</div> <div><div></div> black 99</div>

\* Not available in XXS and XXS PLUS



SIGVARIS

## Half-thigh


This shorter thigh-length stocking is ideal for patients who cannot wear calf-length stockings due to varicose veins behind the knee.

Half-thigh			Toe		Rim	CCL	Colours
TRADITIONAL	A - F 	open			Soft top	2/3/4*	 beige 77
					Knobbed grip top	2/3	

\* Only PLUS sizes

## Thigh with soft top

This option has a particularly soft top and is used on patients with sensitive or allergy-prone skin.

Thigh with soft top			Toe		Rim	CCL	Colours
COMFORT	A - G 	open		closed	Soft top	1 / 2	 savanna 97  caramel 25  black 99
COTTON	A - G 	open		closed	Soft top	1 / 2	 nature 76  black 99
TRADITIONAL	A - G 	open			Soft top	2 / 3 / 4 *	 beige 77

\* Only PLUS sizes













## Thigh with waist attachment

This model is preferred by patients who do not wish to wear a grip top or tights, and it improves quality of life in old age (no tedious donning and doffing of tights for those suffering from age-related ailments such as incontinence, etc.). Can also be worn as a left/right combination.

- Left-hand or right-hand option available
- Belt with a Velcro™ fastener at the waist

Thigh with waist attachment			Toe		Rim	CCL	Colours
<b>COMFORT</b>	A - G 	open	closed		Waist attachment	1 / 2	 savanna 97  caramel 25  black 99
<b>COTTON</b>	A - G 	open	closed		Waist attachment	1 / 2 / 3	 nature 76
<b>TRADITIONAL</b>	A - G 	open			Waist attachment	2 / 3 *	 beige 77



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


\* Only PLUS sizes





## Short lengths

This model is for petite people or people with shorter legs.



EXTRA ASSORTMENT

Short length calf		 Toe 	Rim	CCL	Colours	
COTTON	A - D 	open	closed	Soft top	2	 nature 76  black 99




Short length pantyhose			 Toe 		Rim	CCL	Colours
COTTON	A-T		open	closed	Soft top	2/3	 nature 76



## Pantyhose with compression in pantypart

Bodyform is suitable for customers with soft connective tissue and supports the shape of the hips. Lipoedema around the hips is a further medical indication for compression in the panty area. With bodyform pantyhose the pressure continues up to the pantypart (compression pantyhose does not generally offer any pressure up to the pantypart – the medical compression pressure stops at the thigh).

- Pantyhose with degressive compression pressure in the pantypart
- For perfectly shaped hips

Pantyhose Bodyform		 Toe 	Rim	CCL	Colours	
COMFORT	A-T Bodyform 	open	closed	Soft top	2	<div><div></div> savanna 97</div> <div><div></div> caramel 25</div> <div><div></div> black 99</div>



## Venous ulcers treatment – ULCER X

ULCER X is a compression stocking that offers the following benefits over and above those of bandages:

- Easy to don, meaning that assistance with compression treatment is not required in many instances
- Comfortable and discreet, even with shoes
- Controlled reproducible pressure
- The wound dressing is also secure during the night thanks to the patented under stocking
- No damage to the wound when used correctly
- Kit comprises:
  - 2x understocking (15-20 mmHg)
  - 1x TRADITIONAL (CCL 2)
- Additional understockings are available separately





Venous ulcers treatment		 Toe 	Rim	CCL	Colours	
ULCER X Kit	A - D 	Understocking closed	TRADITIONAL open	Soft top	~45 mmHg	 beige 77

## Post-surgery

This model is suitable for patients who have undergone leg surgery. Available in TRADITIONAL quality for maximum pressure stability throughout the day. (Patients with particularly sensitive skin can also be treated with a COTTON thigh stocking with waist attachment. Understockings can be ordered separately).

- Use for postoperative treatment
- Kit comprises:
  - 2x understocking (without compression)
  - 1x TRADITIONAL with waist attachment
- Additional understockings are available separately

Post-surgery		 Toe 	Rim	CCL	Colours	
TRADITIONAL	A - G 	open		Waist attachment	2 * / 3 *	 beige 77

\* Only PLUS sizes
























## Compression classes 3 and 4

The high compression classes are used in the cases of very advanced chronic venous insufficiency (CVI) and severe lipoedema and lymphoedema.

These patients should be treated by a specialist. When donning and doffing stockings, patients can benefit greatly from our donning aids (see pages 49 - 51).

- In the case of very advanced CVI
- In the case of severe lipoedema and lymphoedema

Compression class 3			Toe		Rim	CCL	Colours
COTTON	A-D 	open	closed	Soft top	3	 nature 76  black 99	
	A-G 	open	closed	Quadrinova Knobbed grip top	3	 nature 76  black 99	
	A-G 	open	closed	Waist attachment	3	 nature 76	
	A-T 	open	closed	Soft top	3	 nature 76  black 99	
TRADITIONAL	A-D 	open		Soft top, Knobbed grip top	3	 beige 77	
	A-F 	open		Soft top, Knobbed grip top	3	 beige 77	
	A-G 	open		Soft top, Knobbed grip top	3	 beige 77	
	A-G 	open		Waist attachment	3 *	 beige 77	

\* Only PLUS sizes

## Compression class 4

Compression class 4		 Toe 	Rim	CCL	Colours	
TRADITIONAL	A - D 	open		Soft top	4	 beige 77
	A - F 	open		Soft top	4*	 beige 77
	A - G 	open		Soft top	4*	 beige 77

\* Only PLUS sizes








## Thrombosis prophylaxis

The optimal soft fabric stocking convinces with its convenient inspection hole for post-surgery thrombosis prophylaxis. The anti-embolism stocking with graduated compression exerts a minimum compression pressure of around 18 mmHg in the ankle region. It must only be used by patients when lying down!

- After surgery (especially abdominal/pelvic and orthopaedic surgery)
- After fractures or injuries to the leg
- Patients with impaired mobility
- For further information please contact your SIGVARIS business partner



Thrombosis prophylaxis		Toe	Rim	CCL	Colours
<b>Anti-embolism stockings</b>	A-D 	With inspection hole	Soft top	~18 mmHg	<input type="checkbox"/> white
	A-G 	With inspection hole	Knobbed grip top	~18 mmHg	<input type="checkbox"/> white
	A-G 	With inspection hole	Waist attachment	~18 mmHg	<input type="checkbox"/> white



SIGVARIS

## Compression arm sleeves

Compression arm sleeves are used to prevent, treat and care for lymphoedema in the upper extremities.

### TRADITIONAL

The yarn's unique stress/strain behaviour guarantees maximum pressure stability throughout the day.

Compression arm sleeve	Hand	Rim	CCL	Colours
<b>TRADITIONAL</b> A-C 	Only mitten	–	–	 beige 77
A-H 	With mitten	With support straps	2	 beige 77
		Knobbed grip top		
C-H 	Without mitten	With support straps	2	 beige 77
		Knobbed grip top		

## Compression arm sleeves

### ADVANCE

The silver-ion-enriched polyamide microfibre has anti-bacterial properties and prevents odours and skin irritations.

- With or without Sensinova grip top
- Bacteriostatic polyamide microfibre
- Thermo-regulating modal fibre

Compression arm-sleeve	Hand	Rim	CCL	Colours
<b>ADVANCE</b>	A - H 	Soft top*	14 - 18 mmHg 20 - 25 mmHg	 light beige 17
		Sensinova grip top		
	C - H 	Soft top*	14 - 18 mmHg 20 - 25 mmHg	 light beige 17
		Sensinova grip top		

\* No PLUS sizes



# SIGVARIS MEDICAL

The CUSTOM MADE assortment

CUSTOM MADE



## The CUSTOM MADE assortment

SIGVARIS offers custom made solutions for all models.


Quality: CCL:	MAGIC 1 + 2	COMFORT 1 + 2	COTTON 1 + 2 + 3	TRADITIONAL 2 + 3 + 4
<b>Models:</b>				
A-D calf with soft top	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
A-D calf with grip top	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
A-F Half-thigh with soft top				<input type="checkbox"/>
A-F Half-thigh with grip top				<input type="checkbox"/>
A-G Thigh with soft top		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
A-G Thigh with grip top	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
A-G Thigh with waist attachment		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
A-T Pantyhose	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
A-T Maternity	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
A-T Bodyform		<input type="checkbox"/>		<input type="checkbox"/>
A-T for man		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
One leg Pantyhose		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>
One leg Pantyhose slip form		<input type="checkbox"/>	<input type="checkbox"/>	<input type="checkbox"/>



Benefits of custom made solutions:

- Perfect fit – tailored specifically to the patient's leg measurements
- Identification of custom made products – the patient's name appears on the label of the stockings

Please don't hesitate to contact your SIGVARIS business partner if you have any questions about the SIGVARIS CUSTOM MADE assortment.

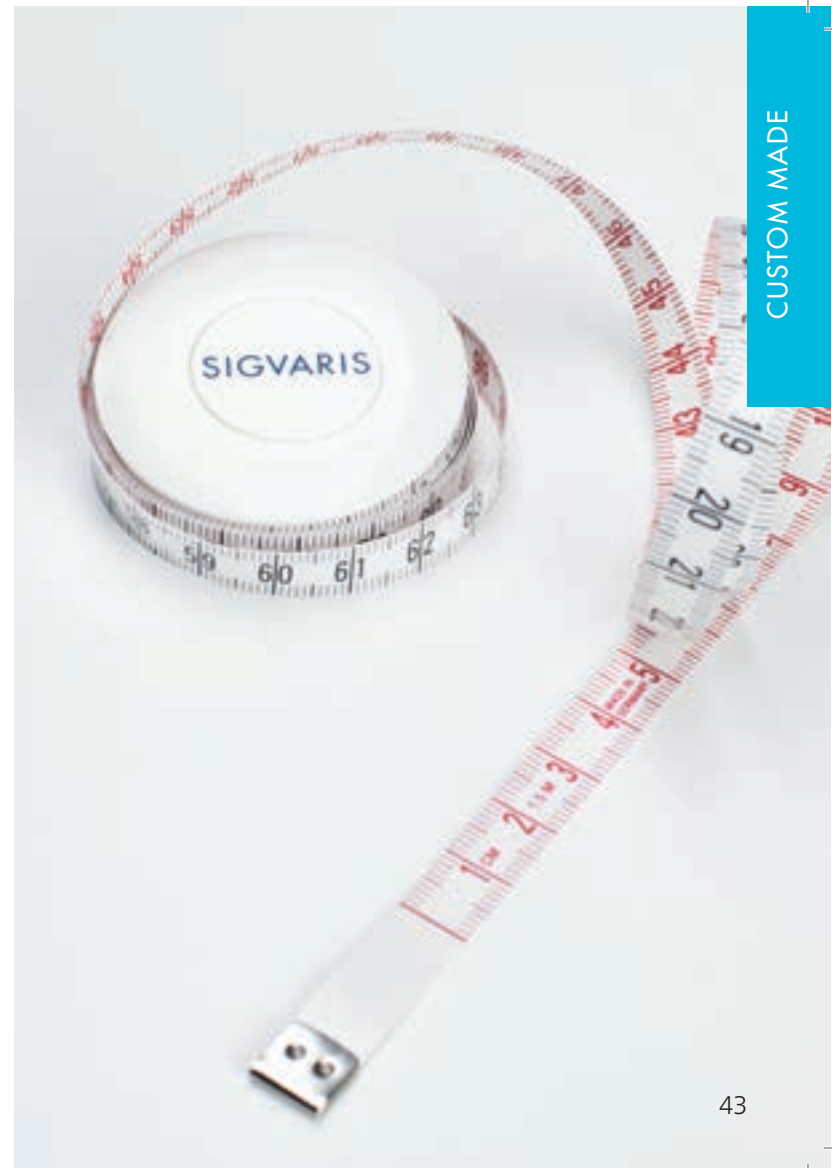



## Taking measurements

Length  $\ell$  in cm;

Circumference  $c$  in cm


### When:

Take measurements first in the morning whenever possible. The leg should be free of oedema/swelling. Take measurements on the skin (not over trousers).


### How:

**Circumference:** Place the measuring tape firmly around the leg/hip/waist without constricting.

**Length:** Measure length at the inside of the leg, with leg straight.

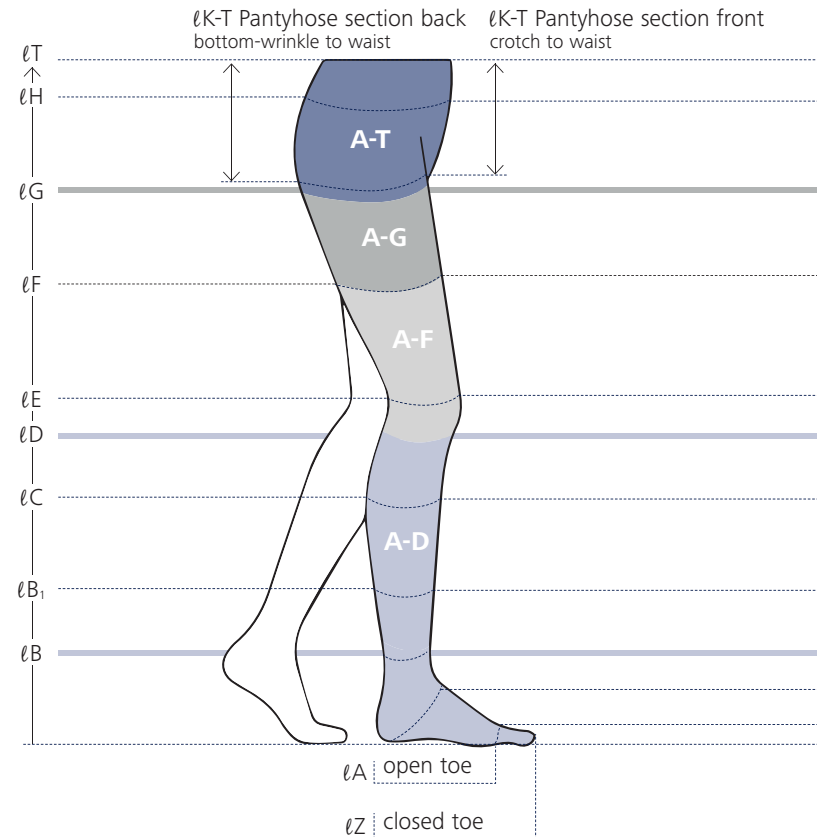

- ..... cT    measured at the waist (top of the hip-bone)
- ..... cH    measured at the circumference of the hip
- cG    measured approx. 5 cm below the crotch
- ..... cF    measured mid-thigh (half-way between patella and crotch)
- ..... cE    measured over the middle of the patella; knee should be slightly flexed
- cD    measured 2 fingers width below knee
- ..... cC    measured widest circumference of the calf
- ..... cB<sub>1</sub>    measured at the beginning of the calf-muscle/exit of Achilles-tendon
- cB    measured at the thinnest part above the ankle
- ..... cY    measured at the heel and instep
- ..... cA    measured at the head of the metatarsus

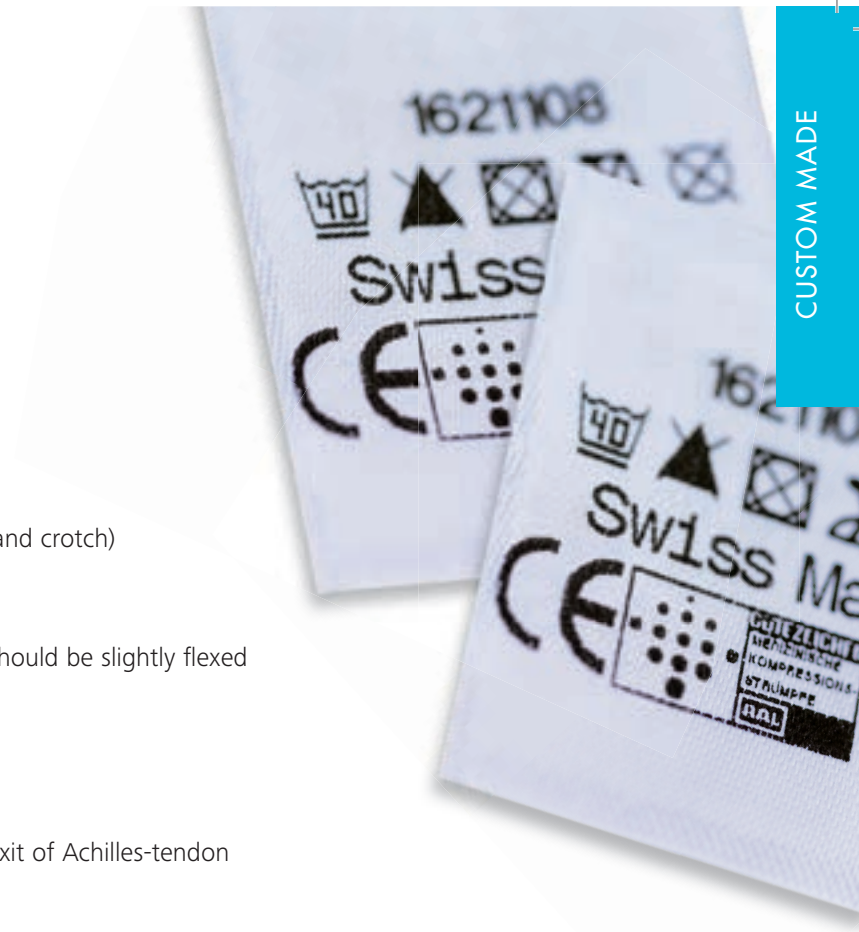

## Alterations

**SIGVARIS will be happy to adapt production and custom made items to the patient's specific needs.**

You can define your requirements when you place your order or at a later date.

Alterations		sew on the narrow grip top (2-3 cm)	sew on the broad grip top (4-5 cm)	sew on the grip top	replace the grip top (an other model)	shorten point of foot (closed toe)	shortening of the pantyhose part	horizontal fly	gusset change	stoma opening	opening in crotch
Models											
A-D Calf	with soft top	■	■			■					
A-F Half-thigh A-G Thigh	with soft top			■		■					
	with grip top				■	■					
A-G Thigh	with waist attachment (left or right)					■	■				
A-T Pantyhose						■	■	■	■	■	■
A-T Bodyform	with compressing pantyhose section					■	■		■		
A-T Maternity	for pregnant women and larger sizes					■	■		■	■	■
A-T for man	with fly (left, right or horizontal)					■	■		■	■	■
Grip top models											
Narrow knobbed grip top (KGT)		■									
Knobbed grip top (KGT)			■	■	■						
Sensinova grip top (SVGT)			■	■	■						

Further changes on request

## Practical aids for everyday life with compression stockings

The SIGVARIS Accessorie assortment includes all the products that make compression stockings more comfortable and easy to wear.

These everyday accessories include donning and doffing aids, care products and washing detergents.

## Donning aids

### DOFF N' DONNER (DND)

The revolutionary donning device for medical compression stockings.

The SIGVARIS DOFF N' DONNER (DND) is a very soft device that allows you to quickly and painlessly guide a compression stocking up and down your leg or arm.

The cone accessory makes it easy to roll stockings onto the SIGVARIS DOFF N' DONNER. The medical compression stocking can also be rolled down easily off the DOFF N' DONNER with the help of the cone.

The SIGVARIS DOFF N' DONNER is perfect for both self-donning and caregiver use.



Cone

DOFF N' DONNER  
(DND)

## Donning aids

### **SIGVARIS sim-slide**

Donning and doffing aid for open-toe compression stockings

- Small (shoe sizes  $\leq 34-37$ )
- Medium (shoe sizes 38–41)
- Large (shoe sizes 42–45)
- X-Large (from shoe size  $\geq 46$ )



### **SIGVARIS magnide on/off**

Donning and doffing aid for closed-toe compression stockings

- Medium (shoe sizes  $\leq 40$ )
- Large (shoe sizes 41–45)
- X-Large (from shoe size  $\geq 46$ )



## Rubber gloves

Rubber gloves make it much easier to don and doff medical compression stockings and tights, allow to better dispose the stocking on the leg, protect the stocking and ensure a longer lifetime.

- Small (glove size 6 1/2 – 7)
- Medium (glove size 7 1/2 – 8)
- Large (glove size 8 1/2 – 9)
- X-Large (glove size 9 1/2 – 10)



## Foot slips

Makes it much easier to don and doff medical compression stockings with an open toe.





## Care products

### Clean Pad

Disposable cleaning cloth with soothing ingredients to improve the grip of the silicone coating on grip tops and improve lifetime. Contains no alcohol.

- 8 small boxes of 10 pads in a box



### Washing Solution (SWS)

Special detergent for medical compression stockings. The washing solution meets all the requirements to ensure the optimum care of your stockings and can also be used for all your delicates.

- 1 single bottle (250 ml)
- 6 bottles in a display box



## Roll-on body adhesive «It Stays»

«It Stays» is a special roll-on adhesive to secure medical compression stockings and arm sleeves without grip top. The adhesive is gentle and supple on the skin. It can also be used to affix orthopaedic and surgical aids to the body. «It Stays» can be washed off with water and leaves no stains.

- 1 piece (60 ml)
- 12 pieces in a display box



## Pressure values of compression classes

Pressure values in the ankle region:

<b>Compression class 1</b>	18 – 21 mmHg
<b>Compression class 2</b>	23 – 32 mmHg
<b>Compression class 3</b>	34 – 46 mmHg
<b>Compression class 4</b>	at least 49 mmHg

## Contraindications

### Relative:

- Oozing dermatoses
- Intolerance of compression stocking material
- Sensory loss of the extremity
- Advanced peripheral neuropathy (e.g. diabetes mellitus)
- Rheumatoid Arthritis (RA)


### Absolute:

- Advanced peripheral arterial occlusive disease
- Congestive heart failure
- Septic phlebitis (inflammation of the vein caused by microorganisms)
- Phlegmasia coerulea dolens



# Sizing system Circumference / Length

## Compression arm-sleeves

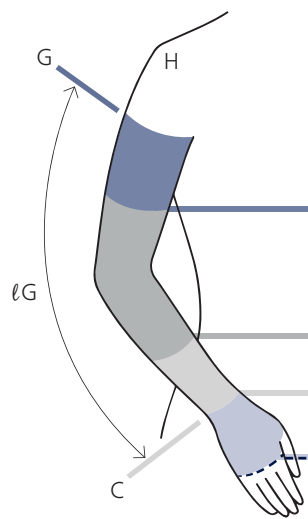



	TRADITIONAL Circumference in cm			ADVANCE Circumference in cm									
	S	M	L	XS	XS PLUS	S	S PLUS	M	M PLUS	L	L PLUS	XL	XL PLUS
<b>cF</b>	24-30	30-36	36-42	24-32	32-38	26-34	34-40	29-37	37-43	32-40	40-46	34-42	42-48
<b>cD</b>	20-24	24-28	28-32										
<b>cC</b>	13-16	16-19	19-22	13-14	13-14	14-16	14-16	16-18	16-18	18-20	18-20	20-23	20-23
<b>cA<sup>1,2</sup></b>	16-19	19-22	22-25	16-20	16-20	17-21	17-21	18-22	18-22	19-23	19-23	20-24	20-24

Length in cm	
short	40-46
long	46-52

Length in cm	
short	<43
long	>43









  



<sup>1</sup> only for mitten (A-C)

<sup>2</sup> only for arm-sleeve with mitten (A-C)

## Available sizes


## TRADITIONAL, ULCER X

		Sizes									Length		
	Compression class	XS	S	S PLUS	M	M PLUS	L	L PLUS	XL	XL PLUS	short	long	x-long
 A-B, Ankle-sock			✓		✓		✓						
 A-D Calf	CCL 2	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	
	CCL 3		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	
	CCL 4		✓ <sup>1</sup>	✓ <sup>1</sup>	✓ <sup>1</sup>	✓ <sup>1</sup>	✓ <sup>1</sup>	✓ <sup>1</sup>			✓	✓	
 A-F Half-thigh	CCL 2	✓ <sup>1</sup>	✓	✓	✓	✓	✓	✓	✓ <sup>1</sup>	✓ <sup>1</sup>	✓	✓	
	CCL 3		✓	✓	✓	✓	✓	✓			✓	✓	
	CCL 4			✓ <sup>1</sup>		✓ <sup>1</sup>		✓ <sup>1</sup>			✓	✓	
 A-G Thigh	CCL 2	✓ <sup>1,2</sup>	✓	✓	✓	✓	✓	✓	✓ <sup>1,2</sup>	✓ <sup>1,2</sup>	✓	✓	
	CCL 3		✓	✓	✓	✓	✓	✓			✓	✓	
	CCL 4	✓ <sup>1</sup>		✓ <sup>1</sup>		✓ <sup>1</sup>		✓ <sup>1</sup>				✓	
 A-G Thigh with waist attachment	CCL 2		✓	✓	✓	✓	✓	✓			✓	✓	
	CCL 3			✓		✓		✓			✓	✓	
 Post-OP Post-surgery	CCL 2			✓		✓		✓			✓	✓	✓
	CCL 3			✓		✓		✓			✓	✓	✓
 A-T Pantyhose A-T Maternity	CCL 2			✓		✓		✓			✓	✓	
 A-T for man	CCL 2			✓		✓		✓			✓	✓	
ULCER X Kit	~45 mmHg		✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	

<sup>1</sup> model only available without grip top <sup>2</sup> model only available in long


Sizing system  
Circumference

**TRADITIONAL, ULCER X**

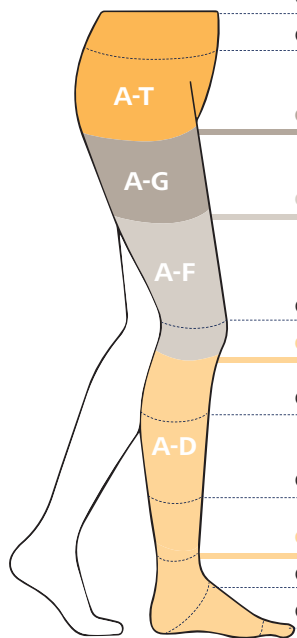



Circumference in cm		XS	S	S PLUS	M	M PLUS	L	L PLUS	XL	XL PLUS
	cT		up to 80	up to 80	up to 90	up to 90	up to 100	up to 100		
	cH		up to 110	up to 110	up to 120	up to 120	up to 130	up to 130		
	cG	40-50	42-52	50-60	50-60	58-68	58-68	66-76	61-71	69-79
	cF	39-48	41-50	45-54	48-57	53-62	55-64	61-70	58-67	64-73
	cE	29-34	32-37	36-41	35-40	40-45	38-43	44-49	41-46	48-53
	cD	26-32	28-34	32-38	31-37	36-42	34-40	40-46	38-43	44-49
	cC	28-31	31-36	35-40	34-39	39-44	37-42	43-48	40-45	47-52
	cB <sub>1</sub>	21-24	24-29	24-29	29-32	29-32	32-36	32-36	35-39	35-39
	cB	17-19	19-23	19-23	23-26	23-26	26-29	26-29	29-32	29-32
	cY	25-28	28-32	28-32	32-35	32-35	35-38	35-38	38-41	38-41
	cA	17-19	19-23	19-23	23-26	23-26	26-29	26-29	29-32	29-32

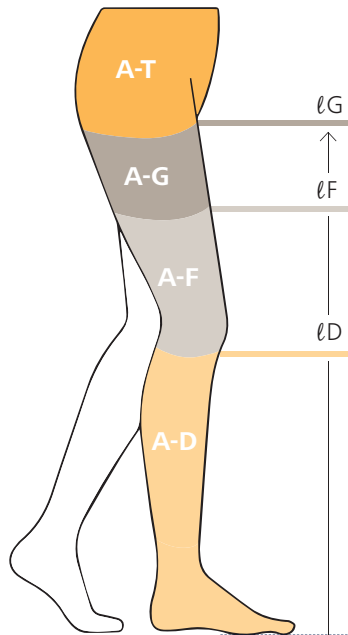A-T = Pantyhose; A-G = Thigh; A-F = Half-thigh; A-D = Calf

## Sizing system Length



## TRADITIONAL, ULCER X

Length in cm							
short	33–37	short	56–60	short	61–70	short	61–70
long	38–44	long	61–70	long	71–80	long	71–80
				x-long*	81–90		
							

\* x-long length only available for TRADITIONAL post-surgery A-G Thigh with waist attachment

Available sizes (p. 57)



Sizing system  
Circumference

**MAGIC, COMFORT, COTTON, JAMES**

Circumference in cm												
	XXS	XXS PLUS	XS	XS PLUS	S	S PLUS	M	M PLUS	L	L PLUS	XL	XL PLUS
cT	60-75	65-80	65-80	75-90	75-90	85-100	85-100	95-110	95-110	105-120	105-120	115-130
cH	75-90	80-95	80-95	85-100	85-100	95-110	95-110	105-120	105-120	115-130	115-130	125-140
without grip top	<b>42-54</b>	<b>47-59</b>	<b>47-59</b>	<b>52-64</b>	<b>52-64</b>	<b>56-68</b>	<b>56-68</b>	<b>61-74</b>	<b>61-74</b>	<b>66-79</b>	<b>66-79</b>	<b>73-85</b>
cG with grip top	<b>42-52</b>	<b>47-56</b>	<b>47-56</b>	<b>52-61</b>	<b>52-61</b>	<b>56-65</b>	<b>56-65</b>	<b>61-71</b>	<b>61-71</b>	<b>66-76</b>	<b>66-76</b>	<b>73-85</b>
A-T												
A-G												
cF	38-49	41-52	41-52	44-55	44-55	49-60	49-60	53-64	53-64	59-70	59-70	63-78
cE	29-36	33-40	33-40	37-44	37-44	39-46	39-46	41-48	41-48	45-52	45-52	46-53
cD	<b>23-30</b>	<b>27-34</b>	<b>27-34</b>	<b>31-38</b>	<b>31-38</b>	<b>34-41</b>	<b>34-41</b>	<b>37-44</b>	<b>37-44</b>	<b>41-48</b>	<b>41-48</b>	<b>42-50</b>
cC	25-32	29-36	29-36	33-40	33-40	35-42	35-42	38-45	38-45	42-49	42-49	43-51
A-D												
cB <sub>1</sub>	22-26	22-26	24-29	24-29	26-32	26-32	29-35	29-35	31-37	31-37	33-39	33-39
cB	<b>18-20</b>	<b>18-20</b>	<b>20-22</b>	<b>20-22</b>	<b>22-24</b>	<b>22-24</b>	<b>24-26</b>	<b>24-26</b>	<b>26-29</b>	<b>26-29</b>	<b>29-32</b>	<b>29-32</b>
cY	27-31	27-31	28-33	28-33	29-35	29-35	31-37	31-37	32-38	32-38	33-39	33-39
cA	18-20	18-20	20-22	20-22	22-24	22-24	24-26	24-26	26-29	26-29	29-32	29-32

A-T = Pantyhose; A-G = Thigh; A-D = Calf